
## Foldable toilet bar Iris

The foldable toilet bar Iris is designed to help you getting up from the toilet. It can be folded upwards if needed. Additionally, a toiletroll holder is provided.

The foldable toilet bar Iris is designed as a general support element. It is useable everywhere(\*), including humid environments.

(\* Only mountable on walls where the wall is suitable to have a bar mounted on. Maximum pulling weight: 115 kg.

Please ensure that the foldable toilet bar Iris is installed according to this manual. If you have any doubts, kindly contact a professional to install the Izzie for you. Use the installation materials which are delivered with the Izzie.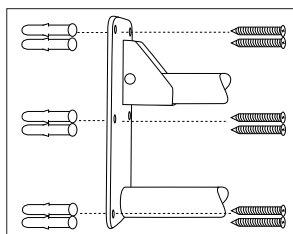

Elements: 1 foldable toiletbar, 6 screws, 6 wallplugs, instruction sheet.


⚠ Avoid hanging heavy objects on the toilet bar as it might cause damage.

After installing, make sure the Iris is firmly mounted. Check on a regular basis if the Iris is firmly mounted. If you notice any defects, refrain from using the Iris.

The foldable toilet bar Iris can be cleaned with normal household cleaning products. Avoid using aggressive cleaning products which can cause discoloration.

Dimensions:

- mounting plate: 23 cm x 10 cm
- total length: 74 cm
- height toiletbar: 17,5 cm
- weight: 1,9 kg

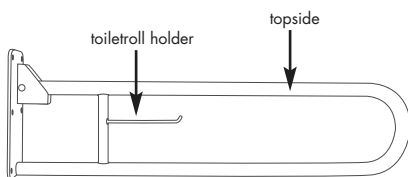Subject to technical alterations. - © 2010

**VERMEIREN**  
Opening up new horizons in mobility

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Vermeiren Suisse S.A.
Vermeiren Nederland B.V.