
Showerbench Kate - Katy

1. Place the showerbench (A) upside down on a working table.
2. Attach the back support (B) by sliding the tubes of the back through the cylinder brackets and lock into place with the snap buttons.

NOTE: The back support is reversible to accommodate left or right, depending on which side you want the armrest.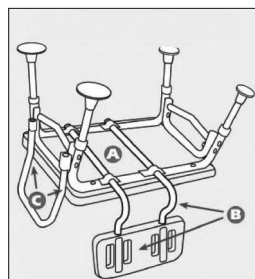


3. Attach the U-shape armrest (C) by sliding it into the mounting brackets and lock into place with the snap buttons.

To adjust the length of the legs: Press the snap button and slide the tube till the desired length. Make sure the snap button is locked into position before using your bench again. Repeat this for the remaining 3 legs.

Attention: Make sure your showerbench is always on an even and smooth surface before usage. Make sure all snap buttons are tightly into place and that all four legs are of the same height.

Model Katy: Insert the toilet bowl through the front side of the showerbench. Ensure that the toilet bowl is correctly positioned before sitting down.

The toilet bowl can be cleaned with normal household cleaning products. Avoid using aggressive cleaning products which can cause discoloration.

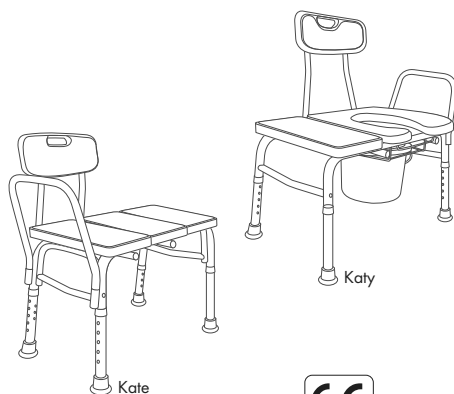Subject to technical alterations. - © 2010

VERMEIREN
Opening up new horizons in mobility

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Vermeiren Suisse S.A.
Vermeiren Nederland B.V.