VERMEIREN

139B

INSTRUCTION MANUAL
MODE D'EMPLOI
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
ISTRUZIONI PER L'USO
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI

Instructions to the specialist dealer

This instruction manual is part and parcel of the product and must accompany every product sold.

Version: A, February 2012

All rights reserved, including translation.

No part of this manual may be reproduced in any form what so ever (print, photocopy, microfilm or any other process) without written permission of the publisher, or processed, duplicated or distributed by using electronic systems.

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INSTRUCTION MANUAL 139B

Technical details

Total height:	950 mm
Height of seat:	525 mm
Width:	570 mm
Seat depth:	420 mm
Length:	820 mm
Weight:	Approx. 14 kg
Max. nominal load:	120 kg

Contents of the consignment

- Frame (incl. commode seat, seat pad, armrests, 4 steering castors, 2 fixing screws for back,
- Backrest
- Footrests
- Toilet bucket with cover
- Instruction manual

Usage

Mobile toilet commode for occasional use at home. Not to be used as a shower stool, for storage, as a ladder or a wheelchair.

Assembly instructions

Position the base-frame in front of you facing forwards. Hold the backrest on the side at the armrest screws (in doing this, the armrests should hang down) and guide the back tubing into the tube openings provided on the base-frame. Turn the locking screws until they are tight.

- ➔ The locking screws must be released, otherwise it may not be possible to insert the back tubing properly.
- ➔ Check that the back tubes are not jammed or skewed and that they are pushed in fully.

Armrests

You can now place the armrests in the receivers provided on the base-frame, by rotating the armrests on their screw fixings, in a curve to the front from behind the backrest. Push the supporting tube of the armrests gently into the receiver.

Footrests

The footrest receivers are to be found on the front frame tubes of the toilet chair. Fold the footplates out from the leg supports, so that you can see which side the relevant footrest should be fitted. The footrests are fitted from the side into the two guide bolts and swung to the centre of the chair until they lock in place.

If this does not happen immediately, gently press again inwards, while gently pressing the retaining mechanism (inner side of footrest hanger). Using the Velcro® fastening, the calf strap can be fitted at the desired position.

Seat pad

The seat pad can only be put in position, if the guide pin is fitted between the commode seat and its hand grip (at the back).

Toilet bucket

The toilet bucket can now be pushed below the seat from the back, with or without its cover.

HINT: If you pour a small amount of water into the upper rim of the toilet bucket, then when the cover is put on, the contents of the bucket will be sealed to prevent bad odours.

Brakes

Before getting onto or off the toilet chair, the parking brakes must be applied to the rear castors. The brakes are applied or released by pushing down or raising the silver brake lever with your foot.

➔ Do not operate the brakes with your hands, as this carries a risk of injury!

Safety

- ➔ Do not stand on the footrests, the toilet chair could tip over – risk of injury!
- ➔ Do not lift the chair by using the armrests or footrests. These parts are removable or can swing aside, so if you used these to lift the chair it could result in injuries or damage.
- ➔ Do not lean too far forwards - danger of tipping over!
- ➔ Repairs: Only by your specialist dealer!
- ➔ Do not reach sideways further than the length of your arm allows – danger of tipping over!
- ➔ Do not use as wheelchair!

Care

The seat pad, arm pads, frame and commode seat can be wiped by using domestic cleaning agents and a damp cloth. The toilet seat should also be cleaned regularly with disinfectant, at least before use is resumed.

Before each use, check screw connections for tightness.

Chrome parts should be wiped periodically with a dry cloth. Use a mild chrome polish on stubborn dirt. Joints should be lightly greased.

➔ Cleaning with hot jets and high pressure equipment is not permitted.

Disinfection

Before the product is handed over, it should be disinfected by an authorised person. Use only disinfectants and verified procedures, in accordance with the recommendations of the Robert-Koch-Institute (disinfection by scrubbing). More information on the website: www.rki.de.

Servicing

At least once a year, your toilet chair should be checked by your specialist dealer, to make sure that the frame, commode seat, pads, armrests and footrests are in good condition, so that your toilet chair remains functional for as long as possible. If these checks are not carried out and documented, then the manufacturer's guarantee and liability is void.

The following maintenance is to be carried out before each use is resumed.

Frame:

Deformation, welded joints, damaged varnish, castors (tracking and fixing screws)

Pads:

Tears, fixing screws

Armrests and footrests:

Deformations, functionality, damage to varnish, fixing screws, greasing, pads

Storage

**between + 5° and + 45° Celsius
20% - 80% relative humidity**

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ENGLISH

CONTRACTUAL WARRANTY

We offer 5 years of warranty on standard wheelchairs, lightweight wheelchairs 4 years. Electronic wheelchairs, tricycles, beds and other products: 2 years (batteries 6 months) and multiposition wheelchairs 3 years. This warranty is limited to the replacement of defective or spare parts.

APPLICATION CONDITIONS

In order to claim this warranty, part "B" of this card has to be given to your official Vermeiren dealer. The warranty is only valid when parts are replaced by Vermeiren in Belgium.

EXCEPTIONS

This warranty is not valid in case of:

- damage due to incorrect usage of the wheelchair,
- damage during transport,
- involvement in an accident,
- a dismount, modification or repair carried outside of our company and/or official Vermeiren dealership,
- normal wear of the wheelchair,
- non-return of the warranty card

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Vermeiren Suisse S.A.
Vermeiren Nederland B.V.