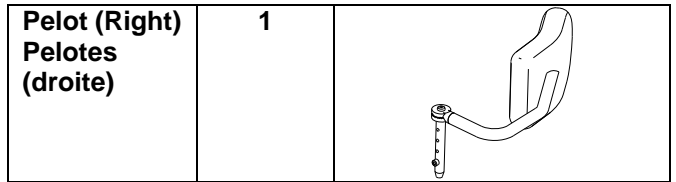
Option:

Side pelots: L04 for backrest L34

Option:

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Some wheelchairs with backrest L34 can be delivered with side pelots: L04. This option gives extra support in the backrest.

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
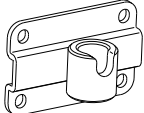
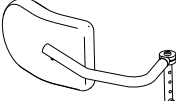
⚠ Check that the side pelots are firmly attached before riding with your wheelchair.

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Description	Quantity	Drawing
Attachments Fixations	2	
Bolt Boulon	8	
Pelot (left) Pelotes (gauche)	1	

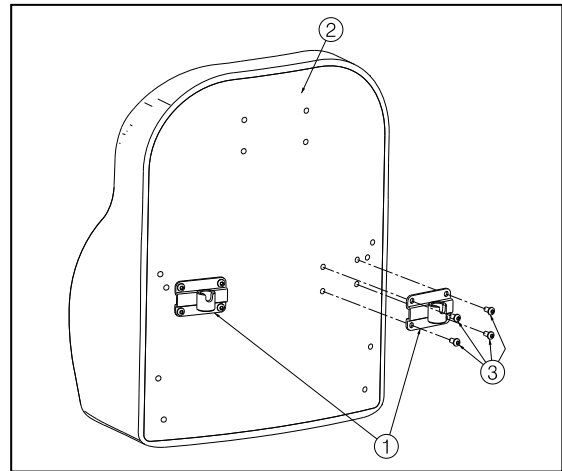4. Assembly / Disassembly / Montage / démontage


The pelot can be mounted by following instructions (steps 1 - 4: only for first time use):

1. Pierce the 8 holes through the upholstery (the holes in the wooden plate are already drilled and tapped).
2. Mount the attachment ① to the backrest ② with the four screws ③.
3. Check that the attachment ① is firmly attached.
4. Repeat for the other attachment.

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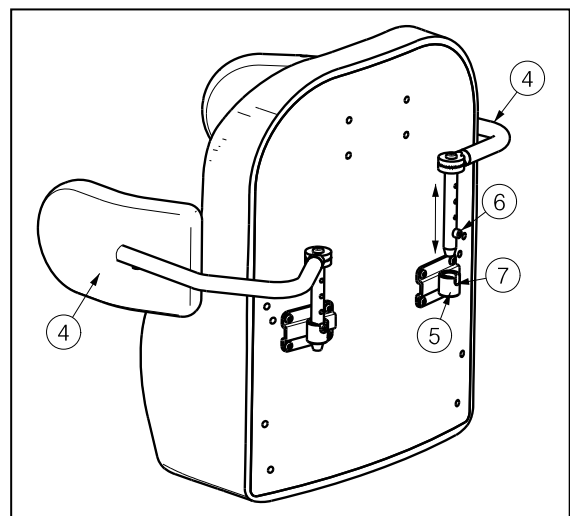5. Grasp the pelot ④.
6. Mount the pelot ④ in the bushing of the pelot attachment ⑤. Check that the bolt ⑥ fits in the slot ⑦.

To remove the pelot:

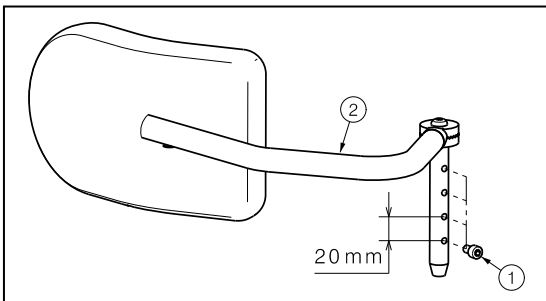
1. Grasp the pelot ④.
2. Pull the pelot ④ out the bushing of the pelot attachments ⑤.

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
The pelot can be adjusted in height by following instructions:

1. Unscrew the bolt ①.
2. Place the bolt ① in the desired hole of the pelot ② to adjust the height (4 positions: increments 20 mm).
3. Retighten the bolt ①.
4. Remount the pelot ② in the attachment (refer paragraph 4).
5. Repeat for the other pelot (check that both pelots are placed in the same height).

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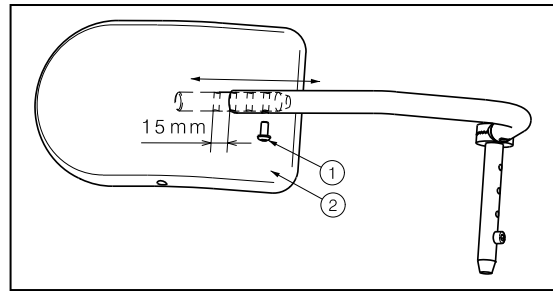
The pelot can be adjusted in depth by following instructions:

1. Unscrew the bolt ①.
2. Move the pelot cushion ② to the desired depth (4 positions: increments 15 mm).
3. Retighten the bolt ①.
4. Repeat for the other pelot (check that both pelots are placed in the same depth).

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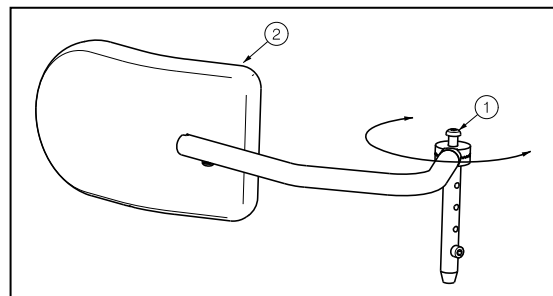The pelot can be adjusted in angle by following instructions:

1. Loosen the bolt ① a little bit.
2. Move the pelot cushion ② to the desired angle (stepless).
3. Retighten the bolt ①.
4. Repeat for the other pelot.

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