



SEAT FRAME SERENYS ZITFRAME SERENYS				Plan No. / Plannr.
				E000173/E
Scale / Schaal: 1:4	Drafted / Getekend	Verified / Nagezien	Confirmed / Bevestigd	
Date / Datum	29/11/2004	26/10/2006	26/10/2006	
Init. / Init.	SD	SD	PJ	

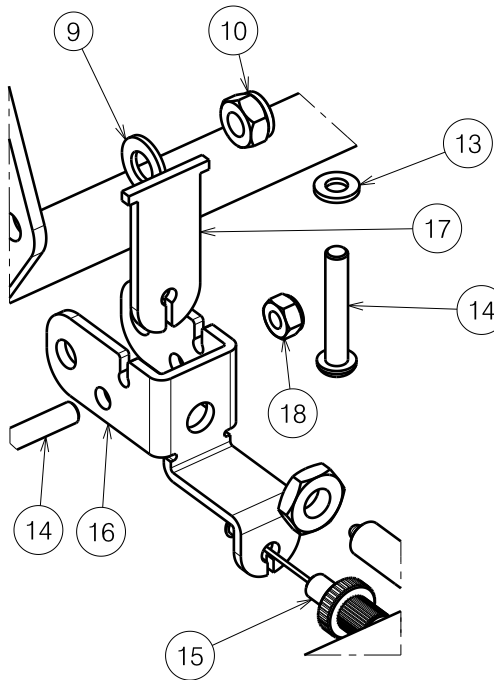

Detail B  
Scale / Schaal 1:2

Detail A  
Scale / Schaal 1:2

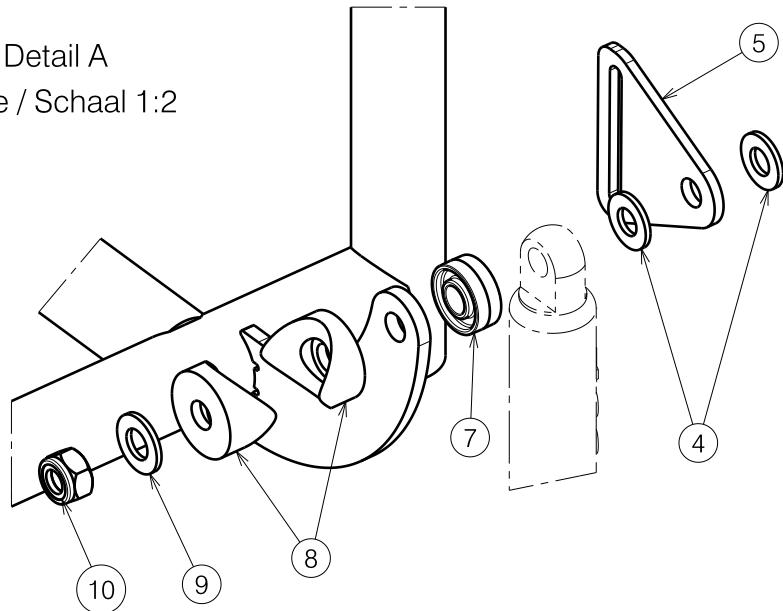

Material / Materiaal			
No. / Nr.	Plan	Article / Artikel	Number / Aantal
1	Y007419 (39)	15865..	1
	Y006765 (45)	15864..	1
	Y007420 (52)	15866..	1
2	---	1007517	1
3	---	1001448	2
4	---	1003199	4
5	Y005495	1607004	2
6	X003646	1999963	1
7	X002480	1993793	2
8	X000106	G003218	4
9	X001119	1001554	4
10	---	1001682	3
11	---	1993594	1
12	---	1005602	1
13	---	1001564	4
14	---	1006057	5
15	---	1900008	1
16	Y011640	1655100	1
17	Y011652	1655200	1
18	---	1001680	1
19*	---	1992267	1

\* not depicted / niet afgebeeld

SEAT FRAME SERENYS ZITFRAME SERENYS				Plan No. / Plannr. E000173/E
Scale / Schaal: 1:4	Drafted / Getekend	Verified / Nagezien	Confirmed / Bevestigd	
Date / Datum	29/11/2004	26/10/2006	26/10/2006	
Init. / Init.	SD	SD	PJ	