


1904560

BACK FRAME SERENYS RUGFRAME SERENYS			Plan No. / Plannr. E000174/K
Scale / Schaal: 1:5	Drafted / Getekend	Verified / Nagezien	Confirmed / Bevestigd
Date / Datum	26/11/2004	19/05/2017	19/05/2017
Init. / Init.	SD	TT	PJ



Material / Materiaal			
No. / Nr.	Plan	Article / Artikel	Number / Aantal
1	Y005819	15974..	1
2	X002921	1996321	2
3	Y005690	15972..	1
4	Y006916	15973..	1
5	X002877 (39)	1996007	1
	X002821 (45)	1996009	1
	X002828 (51)	1996011	1
6	Y007764 (39)	1607162	1
	Y007766 (45)	1607362	1
	Y007768 (51)	1607562	1
7	Y006334	1606904	1
8	M6x40 DIN7984/8.8	1006850	2
9	M6 DIN125A	1001564	8
10	M6x12	1008492	2
11	M5x20	1998368	2
12	X004631	1997303	2
13	VG F30 M6x15	1001577	2
14	M4x12 DIN963/4.8	1005401	4
15	X001052	G066600	2
16	M6x16	1008459	4
17	M4 DIN985	1001678	4
18	M8x25	1995875	3
19	X005513	1992600	2
20	M5 DIN985	1001679	2
21	---	1996806	1
22	Y007779	1607904	1
23	Y003413	1463204	1
24	VG F40 M8x20	1001580	1
25	Y006872	15800	1
26	Y003414	1463304	1
27	M6x20	1006999	4
28	HV8x15	1006661	1
29	VG F30 M6x22	1004392	2
30	X001361	1001674	2
31	M6 DIN985	1001680	6
32	∅22	1990648	2

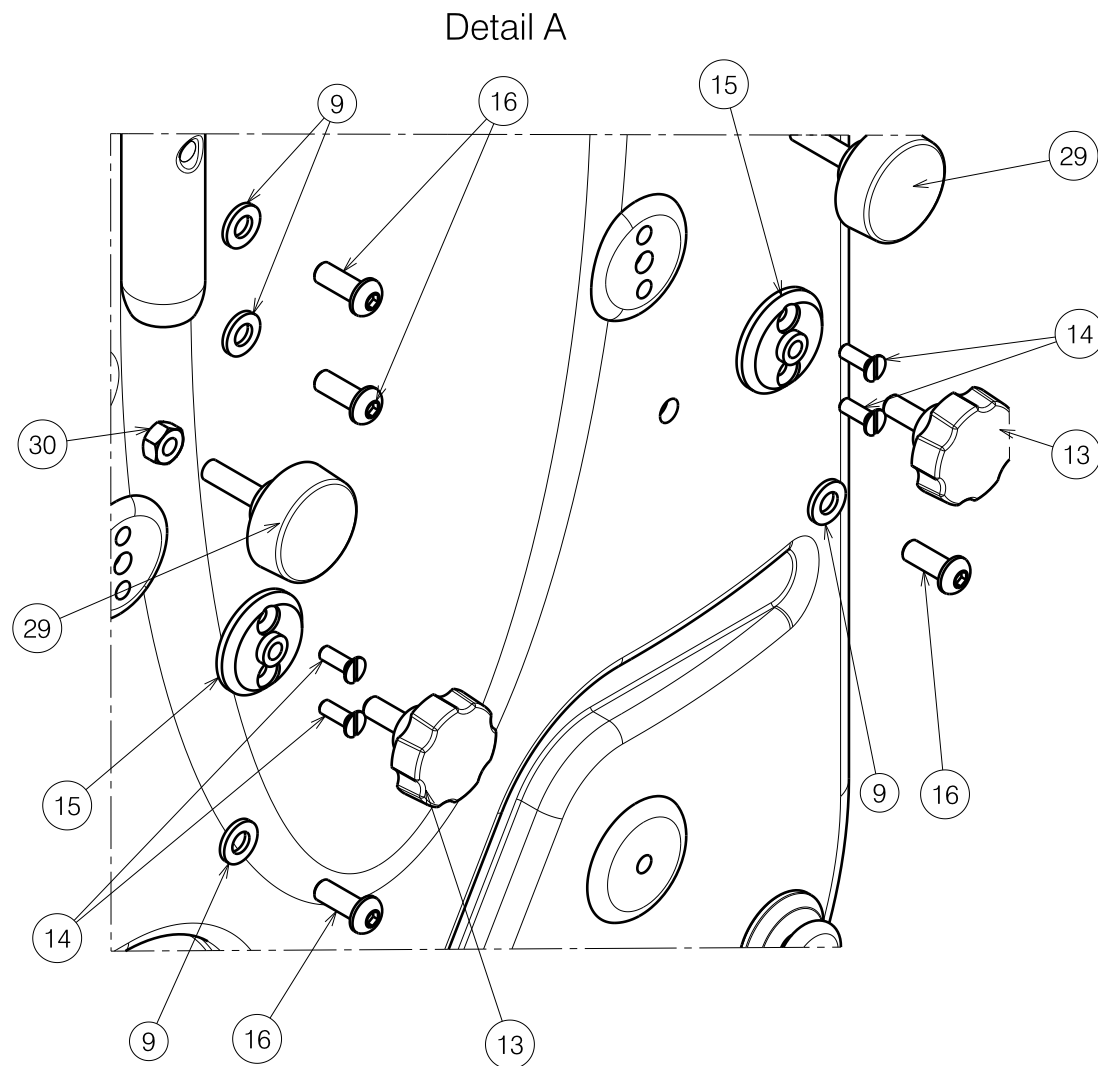

BACK FRAME SERENYS				Plan No. / Plannr.
RUGFRAME SERENYS				
Scale / Schaal: 1:5	Drafted / Getekend	Verified / Nagezien	Confirmed / Bevestigd	
Date / Datum	26/11/2004	19/05/2017	19/05/2017	
Init. / Init.	SD	TT	PJ	